



Synapse Platform User Guide

Prescription analysis modules



Contraindications based on clinical condition module

EXPECTED USE

The “**Contraindications bases on clinical condition**” module promotes the proper use of medications by detecting contraindications and things to be vigilant about based on the patient’s clinical condition.

This module is intended for **healthcare professionals who manage patients’ medications** (doctors, pharmacists, nurses, midwives, dentists etc.).

It provides the contraindications and things to be vigilant about related to a given medication according to the patient’s clinical condition.

The data come from the Thériaque database.

PREREQUISITES BEFORE USE

No training is required before using this module.

The data sheet describes the minimum requirements necessary to operate this module as intended.

HOW TO USE

Fill in the patient’s profile: clinical condition

Enter the patient’s age range, pregnancy/breastfeeding status if applicable, organ failure (kidney, liver, heart), other comorbidities and drug allergies in the “**Patient Profile**”.

N.B. : the stage of kidney failure is calculated according to the Cockcroft-Gault formula using the weight and the creatinine value entered.

Patient Profile

Under 65

From 65 to 75 | Over 75

Pregnancy Nursing

 Kidney failure +

 Liver failure +

 Heart failure +

 Drug allergies +

Contraindications based on clinical condition module

HOW TO USE

Contraindications

Click on the “**Clinical Condition**” tab to view the contraindications and things to be vigilant about related to the patient profile.

Contraindications and things to be vigilant about can be viewed **by medication** **1** or **by clinical condition** **2** .

The alert level (vigilance or contraindication) is given for each situation.

Clinical condition alerts

Note: the results you obtain depend on the information you enter (medicines and patient profile)

1 By drug

By risk factor **2**

OMEPRAZOLE

Kidney disease

- Acute kidney injury **WARNING**

 Alert level

Heart failure

- Uncontrolled heart failure **WARNING**

Drug allergies

CONTRAINDICATION

- Non-steroidal anti-inflammatory drugs (NSAIDS)

In addition to the contraindications, the module gives various alerts such as a risk for driving vehicles, the presence of a doping agent, etc.

Contraindications based on clinical condition module

WARNINGS

The results provided depend directly on **the data entered beforehand** (list of medications, patient profile).

Make sure the information you enter is accurate to get reliable results.

INFORMATION IN THE EVENT OF ANY ISSUES

Please notify us of any complaint or malfunction related to the device at contact@synapse-medicine.com

ADMINISTRATIVE INFORMATION



Manufacturer:

Synapse Medicine

3 rue Lafayette 33 000 Bordeaux

Class I medical device 

Instructions PRO contraindications based on clinical condition module - version v1 (July 2021)