



Synapse Platform User Guide

Prescription analysis modules



STOPP for geriatrics module

EXPECTED USE

The “**STOPP for Geriatrics**” module promotes the proper use of medication by making prescriptions safe for patients over the age of 65.

This module is intended for healthcare professionals who manage patients' medications (doctors, pharmacists, nurses, midwives, dentists, etc.).

It provides information on medications that may be inappropriate for patients over 65 years of age and associated recommendations.

The data come from the second version of the STOPP/START criteria¹

PREREQUISITES BEFORE USE

No training is required before using this module.

The data sheet describes the minimum requirements necessary to operate this module as intended.

HOW TO USE

Once you have entered the medications into the prescription, click on the “**Geriatrics (STOPP)**” button. The STOPP criteria are displayed for each medication concerned.



The screenshot shows a user interface with four tabs: 'Interactions', 'Clinical condition', 'STOPP criteria', and 'Side effects'. The 'STOPP criteria' tab is selected and highlighted with a red box. Below the tabs, the heading reads 'Situations in which the discontinuation of treatment should be considered' with a blue checkmark icon. A note states: 'Note: the results you obtain depend on the information you enter (medicines and patient profile)'. The medication 'Digoxine' is listed as 'DIGOXIN 0,25 mg'. Below this, two red arrows point to the following criteria: 'Acute decompensated Heart Failure with Preserved Ejection Fraction' and 'Digoxin at a long-term dose > 250 µg/day with severe kidney disease (eGFR (estimated Glomerular Filtration Rate) <30ml/min)'.

¹O'Mahony, D., et al., STOPP/START criteria for potentially inappropriate prescribing in older people: version 2

STOPP for geriatrics module

WARNINGS

The results provided depend directly on **the data entered beforehand** (list of medications, patient profile).

Make sure the information you enter is accurate to get reliable results.

INFORMATION IN THE EVENT OF ANY ISSUES

Please notify us of any complaint or malfunction related to the device at contact@synapse-medicine.com

ADMINISTRATIVE INFORMATION



Manufacturer:

Synapse Medicine

3 rue Lafayette 33 000 Bordeaux

Class I medical device 

Instructions STOPP for Geriatrics Module- PRO - version v1 (July 2021)